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## INSTRUCTIONS

### INDEX

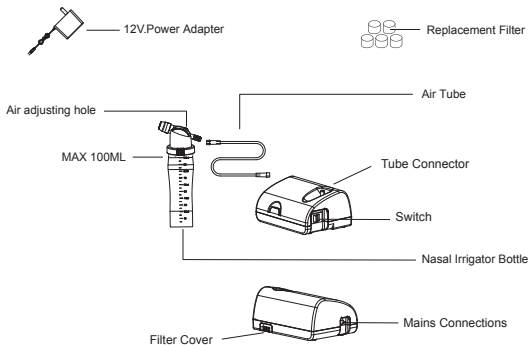
1. Introduction ----- Page1
2. Product Identification----- Page2
3. Important Safeguards----- Page2
4. Operating Your Nasal Irrigator-----Page3
5. Cleaning -----Page4
6. Specifications----- Page5
7. Contact Us ----- Page6

# 1.INTRODUCTION

Thank you for purchasing the Easy Air Mini Sinus irrigation system. Nasal irrigator is used to clean mucus from the nasal passage, making medication more effective. It can also help to clean allergens, irritants, bacteria and viruses from the nose, reducing the frequency of infection as well as help to decrease swelling in the nose and increase air flow.

We encourage you to thoroughly read this guidebook to learn about the features of this product. Any use of this product other than its intended use will void all warranties on this product.

## 2.PRODUCT IDENTIFICATION

## 3.IMPORTANT SAFEGUARD

Note :Read all instructions carefully before use.

The following basic precautions are needed when using an electrical product.

Caution:Failure to read and observe all precautions could result in personal Injury or equipment damage.

## **Product cautions:**

1. To avoid electrical shock:Keep the unit away from water. Do not immerse the power cord or the unit in liquid. Do not use while bathing. Do not reach for a unit that has fallen into water — immediately unplug the unit.
2. Never operate the unit if it has any damaged parts(including power cord). If it has been dropped or submersed in water.
3. The unit should not be used where flammable gas,oxygen or aerosol spray products are being used.
4. Disconnect the unit from the electrical outlet before cleaning,filling and after each use.
5. Do not use other attachments unless recommended by manufacturer.
6. Do not use on children less than 1 year old.

## **Operating Cautions:**

1. Connect this product to an appropriate voltage outlet for your model.
2. Do not leave this product unattended.
3. If any abnormality occurs,discontinue use immediately and call 1-800-915-0116.
4. Always unplug the power plug immediately after use.
5. Never block the air openings of the main unit or place it where the air opening may be obstructed.

## **Storage Cautions:**

1. Store in a dry, cool place.
2. Do not store the unit under direct sunlight, high temperature or humidity.
3. Keep the unit out of reach of children.
4. Always keep the unit unplugged while not in use.

## **Cleaning Cautions:**

1. Do not immerse the unit in water.It will damage the unit.
2. Disconnect the unit from the electrical outlet before cleaning.
3. Clean all necessary parts after each use as instructed in this guidebook.

## 4. OPERATING YOUR NASAL IRRIGATOR

Note : Prior to initial operation, the nasal irrigation bottle and air tube should be thoroughly cleaned.

1. Place your nasal irrigation bottle on a flat and stable surface. Be sure easily reach the controls when you are seated.
2. Gently twist the top part of the nasal irrigation bottle counter-clockwise to disassemble disassemble.
3. Fill the bottom section of the nasal irrigation bottle with the medication prescribed by your physician. (medication not included with unit.)
4. Gently twist the top part clockwise to assemble the nasal irrigation bottle. Be sure the two sections fit well.
5. Attach one end of air tube to the base of the nasal irrigation bottle .
6. Attach the other end of air tube to the air tube connector located on the front of the compressor.
7. Plug the power adapter into an appropriate electrical outlet. Make sure at this stage, the power switch is at "OFF" status.
8. Press power switch to begin your prescribed operation.
9. To perform nasal irrigation , you will flush your nasal cavity by pushing water through one nostril. The water will go behind the nasal septum and come out the other nostril.

NOTE: While performing nasal irrigation , you'll have to tip your head forward and away from the nostril where you are applying the solution. Parental supervision requie for children under 13 years of age.

10. After using, rinse the nasal irrigation bottle with clean water and spray the liquid in the bottle to avoid blocking the spray head.

## **5. CLEANING**

1. Clean and disinfect the irrigation system (including: Nasal irrigation bottle, air tube) after every use.
2. Clean the Nasal irrigation bottle with the distilled water.
3. Completely immersed the nasal irrigation bottle into only approx 75% medicinal alcohol for approx 30 minutes.
4. Wash and rinse the nasal irrigation bottle with about 122°F (50°C) warm distilled water.
5. Shake out excess water. Dry with a clean cloth or leave them to dry on a clean surface.
6. Make sure that the nasal irrigation bottle and other cleaned parts are completely dry, then put all these parts in a dry sealed container.

### **Cleaning the compressor**

1. Wipe daily with a damp cloth.
2. Do not use any powdered cleaners or soap pads, which may damage the unit.
3. Do not immerse in any liquid.

### **Filter change**

1. Do not use cotton or any other materials. Do not wash or clean the filter. Use only Easy Air Mini's filters. And never operate without a filter.
2. Change the filter every 30 days or when the filter turns gray; whichever occurs first.

## 6.SPEIFICATONS

Model:	NI-50
Power:	Power adapter (input AC 100-240V 50/60Hz 350mA, output 12V 1A)
Power Consumption:	≤ 14W
Nasal irrigation cup Capacity:	100ml
Partical Size:	50~90um
Nasal irrigation Rate:	≥35ml/min
Noise Level:	≤49dBA(acc.to DIN EN 13544-1 section 26)
Compressor Pressure Range:	Approx.16~22 Psi/110~151 Kpa
Operation Pressure Range:	Approx.9~16 Psi/61~110Kpa
Operation Temperature Range:	10°C ~to 40°C (50°F ~104°F)
Operation Humidity Range:	10%~ 95% RH
Storage Temperature Range:	-20°C ~to 70°C (-4°F ~158°F)
Storage Humidity Range:	10%~95% RH
Atmospheric pressure:	500~1060hPa
Accessories:	Nasal irrigation bottle(100ml), Air tube(1.6m)
Optional:	Nasal irrigation cap(Silicon)
Dimension:	Approx. L117 X W90 X H58mm
Weight:	Approx. 208g

Protection against electric shock

- Class I Equipment

Protection against harmful ingress of water

-IPXO(Ordinary)

Degree of safety in the presence of flammable anesthetics or oxygen

-No AP/APG(Not suitable for use in the presence of flammable anesthetics or oxygen)

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Home Aide Diagnostics, Inc.